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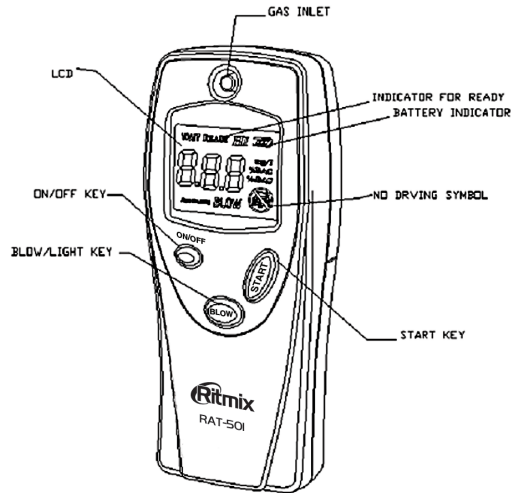
- 1) Install 3 x 1.5V alkaline batteries (AAA) into the battery compartment (auto-cleaning will automatically activates) The **"AUTO CLEANING"** indicator flashes and the display counts down. Press **ON/OFF** Key can terminate this **"AUTO CLEANING"** process.
- 2) Wash your mouth with water before using. After drinking, it is recommended to last after 20 minutes, it is because 20 minutes needed approximately for alcohol to be absorbed into blood from the digestive organs, and residual alcohol remaining in the mouth takes this long to dissipate.
- 3) Pressing the **ON/OFF** Key to turn on the power, sensor pre-heating will be activated for an approximate 15 seconds. **"Wait"** flashes for 9 seconds and then **"PH"** & **"Sta"** shows alternatively and the display counts down.
- 4) Insert a drinking straw or blow tube into the gas inlet located on top of the LCD screen.
- 5) **"READY"** lights with a beep tone when count down to zero, and ready to start the test.
- 6) Press & hold the **"BLOW"** Key, the **"BLOW"** indicator shows on the display. User should blow continuously through the straw for approximately 7 seconds. Stop blowing when you hear a single beep tone and a reading will be displayed on the display.
 - I. A measurement of less than 0.05% BAC will show **"0.0%**"
 - II. A measurement between 0.1% & 1.5% BAC will show result and signs **"HI"** & **"NO DRIVING"** will flash on the display.
- 7) User can press **START** key for another alcohol test. When the display counts down to zero and the **"READY"** indicator shows on the display together with beep tones, it is ready to start another test.

Alert Signals

Ritmix RAT-501 is designed to alert user as following:

- 8) **Low Level** – For measurement below 0.05% BAC, **"0.0%"** will be shown on display
- 9) For measurements between Low Level (> 0.1% BAC up to 1.5%) and the alcohol concentration level shows on the display after test, the analyzer beeps together with **"NO DRIVING"** and **"HI"** flashes for 15 seconds. Press any key can stop the alert sound.

If the results are higher than 1.5% BAC than the signs **"NO DRIVING"** and **"HI"** will be shown on the display. Press any key can stop the alert sound.

⚠ YOU ARE PROHIBITED TO DRIVE since you already exceed the alert level.

Note:

- 1) To ensure high precision readings, user should perform the auto-cleaning if unused for more than 2 weeks (especially for the first time of using). User should start the auto-cleaning process by removing the batteries and power up again. The "AUTO CLEANING" indicator flashes. The 5-minutes auto cleaning starts and the display counts down. Ritmix RAT-501 will give few beep sounds at the last minute of auto cleaning. The analyzer will then turn off automatically after 1 minute and the cleaning process finishes. Before formal test, make sure no alcohol residues left by blowing fresh air into the gas inlet.
- 2) Please notice that pressing the **ON/OFF** key at any time can easily terminate the auto-cleaning process.
- 3) To achieve an accurate reading, user should ensure there is no alcohol residue inside the chamber before starting a new alcohol test. Users can pre-check the tester until the '**LOW**' indication is shown on the display. This may be achieved by pressing **START** key once. Holding the **BLOW** key and blow fresh air into gas inlet for about few seconds. Alternatively, shaking the unit forwards and backwards several times will allow any alcohol residue to leave before starting a new alcohol test.
- 4) After any 4 consecutive tests, allow the device to rest for 2 minutes.
- 5) An error message "**Err**" flashes on the display for 5 seconds for the following improper operations:-
 - i. When user does not hold the **BLOW** key for at least 7 seconds during blowing until a beep sound is heard.
 - ii. When user fails to start the test within 8 seconds after "**READY**" flashes.
- 6) When the product is unused, please store in a dry environment.
- 7) When low battery inductor flashes, the reading is not valid, please replace the batteries immediately.
- 8) If the difference between two successive reading $>0.1\%$, user should test again.
- 9) $0.5\% \text{ BAC} = 0.05\% \text{ BAC} = 0.25\text{mg/l}$

Specification:

1. Range:	0.0 ~ 1.5‰ BAC
2. Power Input:	3 x 1.5 AAA Battery
3. Display:	LCD display with Backlighting
4. Auto Power Off:	2 minutes
5. Breath Sampling:	not less than 7 seconds continuous breath sample
6. Recycle Time:	~30 seconds
7. Dimension:	107 x 45 x 25 mm
8. Weight:	53 g (without battery)
9. Accessory:	3 pcs blowing tubes

Precaution

- After drinking, it is recommended to last after 20 minutes, it is because 20 minutes needed approximately for alcohol to be absorbed into blood from the digestive organs, and residual alcohol remaining in the mouth takes this long to dissipate.
- For accurate result, do not repeat next testing within 3 minutes after last time.
- Do not smoke and eat for 20 minutes prior testing.
- Do not use chemicals to clean the tester directly. For cleaning the mouth piece please use the dry material or paper napkin.
- Do not blowing liquid into the breath pipe.
- Do not use any excitant substances near the tester, such as paint, insecticides, alcohol.
- Do not place the tester in the closed environment with contaminated air.
- Each individual's body responds differently to alcohol. Drink one glass of wine somebody can feel slightly drunken, somebody can feel heavily drunken.
- Diabetes or low-calorie diet or other factors are leading to the rising of the concentration of ketones in exhalation and can cause false positive results.
- Any of Alcohol testers doesn't give the 100% of guarantee of the accurate results. The most accurate results give blood measuring test. That's why if the road patrol has stopped you, and alcohol tester showed the presence of alcohol in blood, require the blood test.
- Reports of World's Health Organization shows that if the alcohol concentration reaches the 0.08 g/ 100 ml, than the risk of road accidents increases in 2.5 times to compare if there is no alcohol in blood. If the alcohol concentration in blood reaches 0.1 g/ 100 ml, risk of road accidents increases in 4.7 times, than there is no alcohol in blood. Even if the slight quantity of alcohol is drunken it increases risk of road accidents in 2 times.
- The unit is designed to be used as an aid to promote reasonable driving. It should not be used in anticipation of the use or operation of vehicles, machinery, or equipment. The unit makes no references whatsoever to blood alcohol content or other such measures of intoxication promulgated by government, medical or other agencies or bodies. The result of this alcohol gives BAC for reference only; manufacturer does not to take any legal responsibility.
- Test results of alcohol tester are not probative.
- Don't drive in a drunk condition of any degree of intoxication.

Important notes

This device can be used only for personal use. The testing results of this device don't have any validity and can't be used in court. Producer and the seller doesn't have any responsibility if there will be the difference in the testing results between which were made by the third party or by other equipment. By buying this device you agree with these conditions.

Warranty

The company-producer guarantees the normal work of device during the all warranty period if the device will be used according to the user manual.

Warranty period for this device is 12 months since the day of the selling through the commercial network. Time of service for this device is not less than 1 year.

The company-producer doesn't accept reclamations in case of the mechanical damage of device, in case of dismantle, wrong usage, or using not in a proper way.

This device will serve you for many years if careful handling.

Company «Ritmix» has the right to change the Specification without notification.

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